Instructions for use

Dräger Alcotest 4000 Go

Download IfU at draeger.com/ifu

en, ja

en	Instructions for use	3
ja	取扱説明書	13

Contents

1	Safety-related information.....	4
2	Conventions in this document.....	4
2.1	Meaning of the warnings.....	4
2.2	Typographical conventions.....	4
2.3	Trademarks.....	4
3	Description	4
3.1	App startup screen.....	4
3.2	Product overview	5
3.3	Intended use	5
3.4	Symbols	6
4	Use.....	6
4.1	Prerequisites for use	6
4.2	Starting the device for the first time	6
4.3	Taking a breath sample	7
4.4	Confirming the test results	8
4.5	Test history	8
4.6	Turning off the device	9
4.7	Removing the mouthpiece	9
4.8	Disconnecting the device.....	9
4.9	User menu	9
4.10	App menu items	9
5	Troubleshooting.....	10
5.1	Resetting the device	10
6	Maintenance	10
6.1	Calibration.....	10
6.2	Changing the battery.....	10
6.3	Cleaning the device	11
7	Storage.....	11
8	Disposal	11
9	Technical data	11
10	Annex	12
10.1	Glossary.....	12

1 Safety-related information

- Before using this product, carefully read these instructions for use and those of the associated products.
- Strictly follow the instructions for use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the intended use section of this document.
- Do not dispose of the instructions for use. Ensure that they are stored and used appropriately by the user.
- Do not use a faulty or incomplete product. Do not modify the product.
- Notify Dräger in the event of any component fault or failure.
- Comply with all local and national rules and regulations associated with this product.
- Only trained and competent personnel are permitted to inspect, repair and service the product. Dräger recommend a Dräger service contract for all maintenance activities and that all repairs are carried out by Dräger.
- Use only genuine Dräger spare parts and accessories. Otherwise, the proper functioning of the product may be impaired.
- Only use the accessories and components described in these instructions for use and/or those approved by the manufacturer.
- Portable RF communication devices (radios), including their accessories such as antenna cables and external antennas, must not be used within 30 cm (12 in) of the device. Non-compliance can lead to lower device performance. An exception to this rule is the smartphone.
- Only use batteries that Dräger has approved for this product (see "Technical data", page 11).
- Do not expose the device to direct sunlight, as this leads to a rise in temperature beyond the indicated limit values of the ambient conditions (see "Technical data", page 11).
- Do not use the device while driving.
- These instructions for use can be downloaded in electronic form from the Technical Documentation database (www.draeger.com/ifu) or obtained free of charge as a printed copy from Dräger.

 The device was fully cleaned and disinfected before delivery.

2 Conventions in this document

2.1 Meaning of the warnings

The following warnings are used in this document to alert the user to possible hazards. A definition of the meaning of each warning is as follows:

Warning symbols	Signal word	Classification of warning
	WARNING	Indicates a potentially hazardous situation. If not avoided, it could result in death or serious injury.

Warning symbols	Signal word	Classification of warning
	CAUTION	Indicates a potentially hazardous situation. If not avoided, it could result in physical injury. It may also be used to alert against unsafe practices.
	NOTICE	Indicates a potentially hazardous situation. If not avoided, it could result in damage to the product or environment.

2.2 Typographical conventions

Text	Text in bold identifies labels on the device and text on the display.
	This triangle in warnings identifies the options available for avoiding the hazard.
	The greater-than symbol identifies a navigation path in a menu.
	This symbol indicates information that may make the use of this product easier.

2.3 Trademarks

Trademark	Trademark owner
Bluetooth®	Bluetooth Special Interest Group (SIG)
Alcotest®	Dräger Safety AG & Co. KGaA


The following web page lists the countries in which Dräger's brands are registered: www.draeger.com/trademarks.

3 Description

3.1 App startup screen

Overview of startup screen:


 Differences between Android and iOS are possible.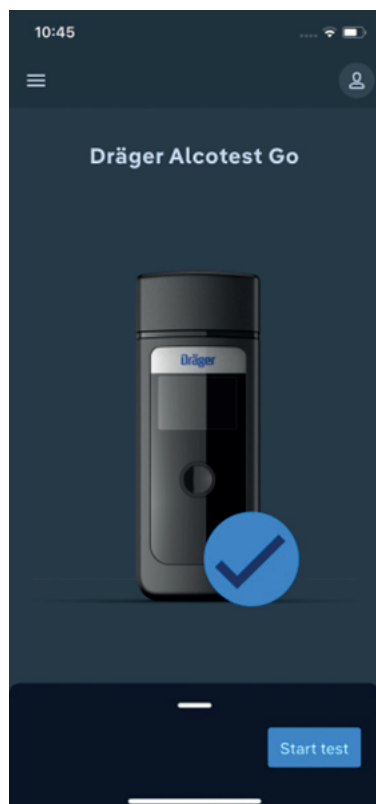

- Menu access
- Access to user menu
- Box at the bottom of the screen:
 - Information area:
e.g. **Connect device**, **Start test**
 - Test history: Drag box upwards

3.2 Product overview

Parts of the device: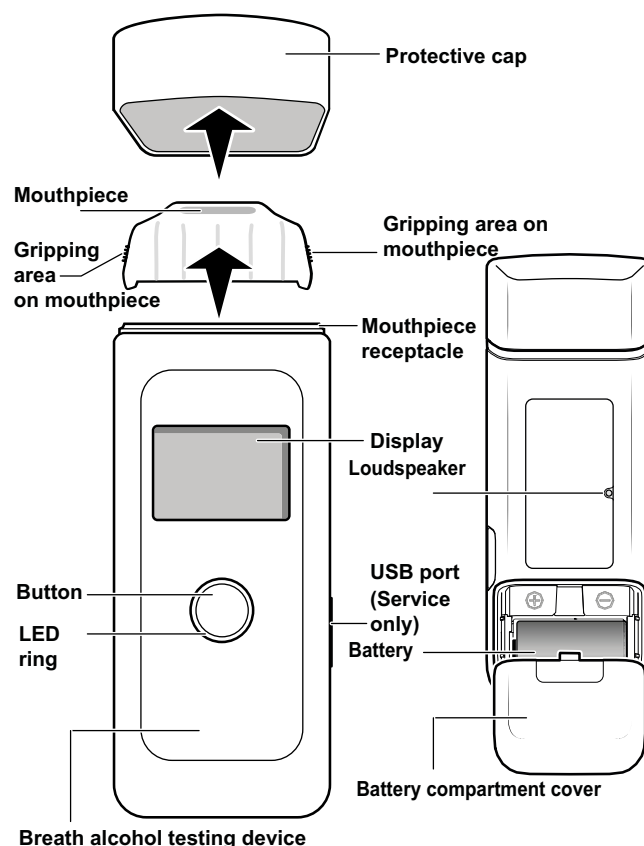

3.3 Intended use

Dräger Alcotest 4000 Go

The breath alcohol testing device measures alcohol content in human breath using an electrochemical sensor. This is a test device with active sampling.

The device has a BLE interface that ensures communication with the app. The determined breath alcohol concentration is transmitted to the app via the interface. No therapeutic measures should be derived from the measurement results.

Mouthpiece 3820

To fulfil the intended purpose, the mouthpiece must be connected to the device.

Contraindications

- Residual mouth alcohol
Residual mouth alcohol means that the breath alcohol concentration (BrAC) measured by blowing does not yet match the blood alcohol concentration (BAC). The alcohol consumed has not yet been fully distributed in the blood. The residual mouth alcohol usually takes 15 minutes to subside.
Residual alcohol in the mouth can distort the test results.
- False-positive test result
Certain foods can lead to a positive test result. Aromatic drinks (e.g. fruit juice), alcoholic mouth sprays and medicines, among other things, can distort the test results.

3.4 Symbols

Device label

	Observe the instructions for use
	Upper and lower temperature limits The storage temperature is indicated on the packaging. The utilisation temperature is indicated on the device.

Device display

	Device with mouthpiece
	Bluetooth®
	Battery

4 Use

4.1 Prerequisites for use

- Observe a wait time of around 15 minutes after the last food intake (eating or drinking, e.g. alcohol).
- The wait time cannot be shortened by rinsing the mouth with water or a non-alcoholic drink.
- Residual mouth alcohol or aromatic drinks (e.g. fruit juice), alcoholic mouth sprays and medicines can distort the measurement result.
- Alcoholic disinfectants (e.g. hand sanitizers) can distort the measurement result.
- Belching and vomiting can distort the result.
- Only carry out the measurement if safe to do so under the circumstances, e.g. not while driving a car.
- Breathe regularly and calmly before sampling.

4.1.1 App installation

To be able to use the device, the app must be installed and initialised from the Apple App Store or Google Play Store. For initialisation, follow the instructions provided by the app. Details on how to initialise the app can be found in the relevant chapter. (see "Initialising the app", page 6)

4.1.2 Preparing the device for use

WARNING

Risk of injury

Sharp edges on the battery compartment can cause injuries.

- Do not use the device without the battery compartment cover.

1. Make sure that the device batteries (see "Technical data", page 11) are correctly inserted and that they are charged.
2. Make sure that a clean mouthpiece has been fitted on the device.

4.1.3 Device navigation

1. Press the button until the desired entry is reached.
2. To confirm selection, press and hold the button for approx. 2 seconds.

4.2 Starting the device for the first time

4.2.1 Initialising the app

1. On a smartphone, open the Apple App Store or Google Play Store.
 2. Download and open the app.
 3. The licence terms are displayed when first starting up the app and afterwards in the legal notices (see "Terms & Conditions, Legal notice, Privacy Policy, Third Party Licensing", page 9). If the terms are not accepted, the app cannot be used.
 4. To sign into the app, select the **Login** button and enter the email address and the initial password.
This step can be skipped by selecting the **Skip** button.
 5. The app also provides the option of sending data. To send data, you need to sign in via the (see "Log in or Log out", page 9) user menu.
- ✓ The app has been initialised.

4.2.2 Country wizard of the device

The country wizard is opened when the device is first started. The device language can be set here.

1. Briefly press the button to select the desired language.
2. To confirm the selected language, press and hold the button for approx. 2 seconds.

✓ The language is set on the device.

4.2.3 Attaching the mouthpiece to the device

The mouthpiece can be used multiple times for the same test person. If the user changes, a new mouthpiece is also required.

WARNING

Risk of infection

The mouthpiece may be contaminated.

- The mouthpieces are individually packed for hygienic reasons. Do not use mouthpieces from damaged packaging.

WARNING

Risk of choking

Parts of the packaging may remain on the mouthpiece.

- Completely remove all the packaging material.

WARNING

Small parts of the device can come loose and be swallowed.

- Keep the device away from children.

WARNING


Risk of infection (risk of reinfection and cross-infection)

Droplets or contamination can lead to infection.

- Inspect the mouthpiece for droplet formation, contamination or defects before every use and replace if necessary.

 Only the specified mouthpiece (order no. 3325250/5 pcs.) may be used.

1. Remove the protective cap from the device.

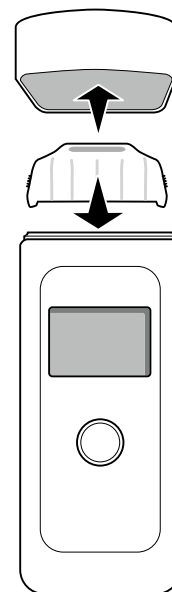2. Remove the mouthpiece from its packaging.
 3. Hold the mouthpiece by the grip zones at the sides.
 4. Insert the mouthpiece into the mouthpiece receptacle at the top of the device. It can only be attached in one direction.
- ✓ The mouthpiece is attached to the device.

4.2.4 Turning on the device

⚠ WARNING

Risk of other devices malfunctioning

When the device is turned on, the Bluetooth® feature is automatically activated.

- Do not turn the device on when travelling in aircraft.

📄 The device was fully cleaned and disinfected before delivery.

📄 The time in the device automatically synchronises with the time on the smartphone.

1. Press the button and hold for approx. 1 second.
- ✓ The device is turned on.

4.2.5 Connecting the device

The app can be connected to a device via the **Connect device** button. Alternatively, this can be done via the menu item **Device connection**.

4.2.5.1 Connecting the app to the device

If the app is not yet connected to a device, proceed as follows:

1. Open the app on the smartphone.
2. In the box at the bottom of the screen, tap **Connect device**.

📄 The access authorisations for Bluetooth® and the device location must first be granted once on the smartphone. If the authorisation is revoked, access must be authorised again.
To establish a connection with the device, Bluetooth® and the location services must be activated.

3. To turn on the device, press and hold the button for 1 second.

4. To start searching for a device, tap **Continue** in the app.

⇒ A device usage information screen appears.

5. Tap **Continue** four times in total.
 - ⇒ **Scanning for device** is displayed and the app searches for compatible devices. After a successful search, the devices found are displayed.
 6. Select the desired device.
 - ⇒ A Bluetooth® connection request appears.
 7. Compare the code displayed on the device with the passkey shown in the pairing request.
 - ⇒ If they match, select **Pair** in the app and tap the button on the device to confirm.
 8. Tap **OK** in the app.
 - ⇒ The app and device are connected with each other.
- ✓ Setup is complete.

4.3 Taking a breath sample

⚠ WARNING

Distorted measurement results


Residual mouth alcohol or aromatic drinks (e.g. fruit juice), alcoholic mouth sprays and medicines can distort the test results.

- Observe the conditions for use (see "Prerequisites for use", page 6).

1. Open the app on the smartphone.
2. Remove the protective cap from the device.
3. If no mouthpiece has yet been attached, take a mouthpiece out of the packaging and attach it to the device. For further information see: "Attaching the mouthpiece to the device", page 6.

i To be able to upload data, log in before taking the breath sample. When logged in, the name is displayed on the startup screen.

4. In the app, tap **Start test** in the box.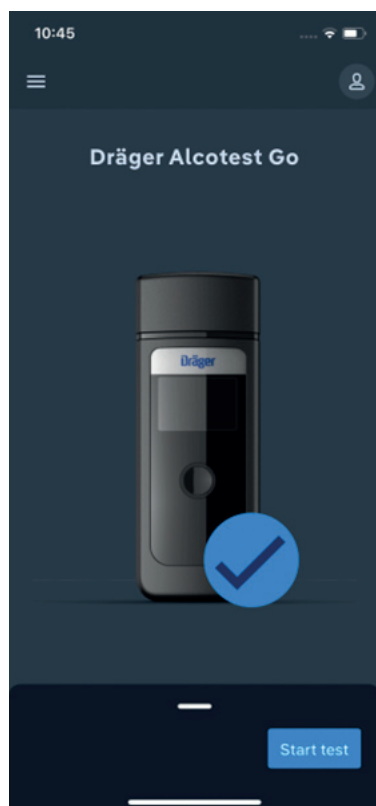


5. To turn on the device, press and hold the button for 1 second.
6. Wait for the app and device to connect.

i The access authorisation for the camera and Bluetooth® must first be entered once. If the authorisations are revoked, access must be authorised once again.

⇒ The app activates the front camera and requests a breath sample.

7. Hold the smartphone in one hand or place it somewhere and hold the device in the other hand.
8. Hold or place the smartphone so that the face and the device can be seen during the breath sample.

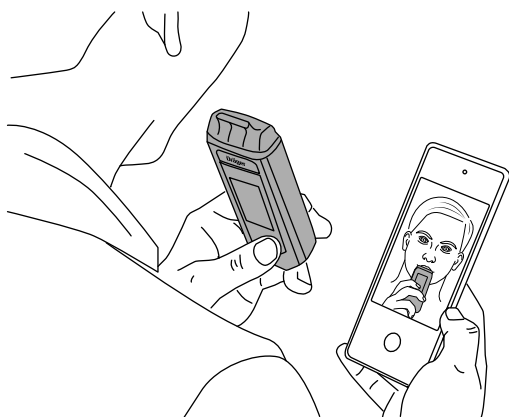9. Continue administering the breath sample until a click can be heard.
- ✓ The photo is taken automatically during the breath sample.
The test result is displayed in the app.
The device can be turned off, or it turns off automatically after a few seconds.

4.4 Confirming the test results

i The user must interpret the test results.

Three test results are possible:

4.4.1 Negative test result

No alcohol is detected in the breath sample.

i The actual breath alcohol concentration can be above the threshold even if a negative test result is displayed.

i After a negative test result, the display automatically returns to the start page after 3 seconds without confirmation.

1. Confirm the test result with **OK**.

4.4.2 Test result below threshold

The measured value is below the defined threshold.

i The test result can only be displayed if a threshold has been defined.

1. Confirm the test result with **OK**.

i A wait time of 5 minutes is required before giving another breath sample.

4.4.3 Positive test result

Alcohol is detected in the breath sample.

i This may potentially also be a false-positive test (see "Contraindications", page 5).

1. Confirm the test result with **OK**.

⇒ An information screen with a countdown appears.

i A wait time of 5 minutes is required before giving another breath sample.

4.5 Test history

All events and results are displayed in the history.

4.5.1 Display test history

The test history can be accessed via the startup screen. To do so, swipe and drag the box at the bottom of the screen upwards. Alternatively, the test history can be accessed via the menu.

The following contents are displayed in the test history:

- **[Test result] mg/L** (or other measurement module)
- **Device paired**
- **Device unpaired**
- **Technical problem**

4.6 Turning off the device

 The device is disconnected after the test and the device automatically turns off after a few seconds. If the app is closed, which also disconnects the device, the device automatically turns off after a few seconds.

1. Press and hold the button.
- ✓ The device turns off after 3 seconds.

4.7 Removing the mouthpiece

1. Pull the mouthpiece up and off the mouthpiece receptacle.
2. Dispose of the mouthpiece in compliance with local regulations.

4.8 Disconnecting the device

The menu item **Device connection** can be used to disconnect the app from a device.

4.8.1 Removing an existing device

To disconnect an existing connection between the app and device, proceed as follows:

1. Open the app on the smartphone.
2. Turn on the device by pressing and holding the button for 1 second.

 The access authorisation for Bluetooth® must initially be issued once. If the authorisation is revoked, access must be authorised again. To establish a connection with the device, Bluetooth® and the location services must be activated.

3. In the app, use the following menu path: **Menu > Device connection**
 4. Tap **Unpair** in the app.
 5. Confirm that the device has been disconnected by tapping **Confirm** in the app.
- ✓ The app and device are disconnected.

4.8.2 Removing a lost device

If the device is lost, the connection can be disconnected as follows:

1. Open the app on the smartphone.
2. Follow this menu path: **Menu > Device connection**
3. Click on **Unpair**.
4. Confirm that the device has been disconnected by tapping **Confirm**.

5. Confirm that the lost device has been disconnected by tapping **Confirm**.

✓ The app and device are disconnected.

4.9 User menu

Settings can be configured and help functions can be accessed in the user menu. The user can also sign in and sign out here.

4.9.1 Settings

The settings show the user ID (of the signed-in user) and the following settings can be configured in the app:

- **Delete data after 28 days**
If this feature is activated, all entries saved in the test history are deleted after 28 days.
- **Last synchronization:**
This displays when data was last synchronised.

4.9.2 Help

For technical errors or problems, a Support phone number and email address is displayed and the instructions for use are stored here.

4.9.3 Log in or Log out

 Data can only be transmitted if the user has signed in before the breath sample.

 After signing in, the user's given name, surname and email address are displayed in the user menu.

The user can sign in or out here.

After signing in, the user's test results are transferred and displayed in the test history. All data created before signing in is no longer visible in the test history.

After signing out, the user's test results will no longer be displayed in the test history until they sign in again.

4.10 App menu items

4.10.1 Device connection

The app and the device can be connected or disconnected here.

4.10.2 Test history

This displays the test history.

Among other things, the test history shows the results of the breath samples.

4.10.3 Terms & Conditions, Legal notice, Privacy Policy, Third Party Licensing

Hyperlinks to the following documents can be found here: *General Terms & Conditions, Legal Notice, Privacy Policy and Third-Party Licensing*.

5 Troubleshooting

If the device malfunctions, an error message will appear. The number that appears below the message is for service purposes. If the error persists even after turning the device on and off multiple times, contact Support (see "Help", page 9).

Dräger Alcotest Go (app)

Malfunction/Error	Cause	Remedy
The results are not transmitted.	The Bluetooth connection is missing.	Check the Bluetooth connection on the smartphone.

Dräger Alcotest 4000 Go

Malfunction/Error	Cause	Remedy
Device cannot be turned on.	The battery is flat. The battery is not correctly inserted.	Change the battery. Check whether the battery is correctly inserted.
Volume too small/sample invalid	The test person is not blowing strongly enough or with interruptions.	Repeat the breath sample. Blow more forcefully or without interruption into the mouthpiece.
Volume too big	The measuring range has been exceeded.	Repeat the breath sample.
Device outside temperature range	The device is too hot or too cold.	Allow the device to cool down or warm up.
Sample interruption	Irregular exhalation, abrupt ending or inhalation	Repeat the breath sample.
Error number	Device error	Turn the device off and on again. If the error persists, contact Support.
Calibration interval expired	The calibration interval has expired.	Contact Support. Have authorised personnel carry out the calibration.
Bluetooth® connection not possible	The device is not turned on or the battery is flat.	Turn on the device or change the batteries.
Bluetooth® connection not possible	The smartphone is too far away from the device or other objects are located in between.	Place or hold the smartphone and the device directly next to each other when establishing a connection.
Bluetooth® connection not possible	The device is connected to a lost smartphone.	Contact Support.
Bluetooth® connection not possible	The device was disconnected in the smartphone's Bluetooth® settings.	Contact Support.

Dräger Alcotest 4000 Go

Malfunction/Error	Cause	Remedy
Bluetooth® connection not possible	A new device has been received.	Disconnect the lost device (see "Removing a lost device", page 9) and connect the new device with the app (see "Connecting the app to the device", page 7).

5.1 Resetting the device

If a Bluetooth® connection cannot be established, or the smartphone with the connected A4000 Go is missing, the device will need to be reset.

1. Start the device.
 2. Press and hold the button for 3–4 seconds, until the shut down timer starts.
 3. Quickly press the button four times in succession.
 4. Press and hold to confirm the connection of a new smartphone.
⇒ The device displays the configuration wizard.
 5. Set the language, date and time.
- ✓ The device is reset and ready to be connected to the smartphone.

6 Maintenance

The device must be calibrated and serviced annually. Only authorised persons may calibrate and service the product. Contact Support.

 Comply with the national guidelines governing calibration.

6.1 Calibration

The device contains a calibration counter. The device indicates that the calibration is due to expire 30 days before the expiration of the calibration interval.

When the device displays the remaining term of the calibration, contact Support.

6.2 Changing the battery

WARNING

Damage to the device

The batteries cannot be charged via the USB port. The USB port is for Service only.

► Do not charge the batteries using the USB port.

 Only use leak-free batteries in line with UL that have a  symbol.

1. Turn off the device.
2. Slide the battery cover off.
3. Change the battery (see "Technical data", page 11).
4. Make sure that the battery is correctly inserted.
5. Slide the battery cover back on.

6.3 Cleaning the device

⚠ WARNING

Damage to the device

The ingress of liquids into the device can damage the device.

- ▶ Do not hold the device under flowing water.
- ▶ Make sure that no liquid enters the device openings.

⚠ WARNING

Distorted measurement results

Cleaning agents can distort the measurement results.

- ▶ Do not use any cleaning agents.

i The device was fully cleaned and disinfected before delivery.

1. Turn off the device.
2. Place the protective cap on the device.
3. Only wipe down the device with a cloth moistened with luke-warm water.
4. Make sure that a clean mouthpiece has been fitted.

7 Storage

⚠ WARNING

Damage to the device

Leaking battery acid can cause damage to the device.

- ▶ Remove the batteries if the device is not used for a prolonged period of time.

8 Disposal

This product must not be disposed of as household waste. This is indicated by the adjacent symbol.

You can return this product to Dräger free of charge. For information please contact the national marketing organizations or Dräger.

Batteries must not be disposed of as household waste. They are therefore marked with the adjacent symbol. Dispose of batteries at battery collection centres as specified by the applicable regulations.

9 Technical data

Dräger Alcotest 4000 Go

Ambient conditions

During operation	-5 to +40 °C (23 °F to 122 °F) >15 to <95 % r.h. (non-condensing) 600 to 1300 hPa
During storage and transport	-20 °C to +60 °C (-4 °F to +140 °F) >15 to <95 % r.h. (10 to 100 % r.h. <48 h)

Measuring principle

Electrochemical sensor

Measuring range

Dräger Alcotest 4000 Go

Breath alcohol concentration	0.00 to 2.5 mg/L (mass of alcohol per minimum volume of breath at 34 °C and 1013 hPa)
Blood alcohol concentration	0.00 to 5.00‰ (mass of alcohol per blood volume or mass of blood at 20 °C and 1013 hPa)

When providing these data, a conversion factor of 2100 between the breath and blood alcohol concentration is used.

Example: mg/L in ‰ 1 mg/L x 2.1 = 2.1 ‰	Example ‰ in mg/L 1 ‰ / 2.1 = 0.476 mg/L
--	---

Sampling

Minimum volume	>1.2 L
Breath flow	>10 L/min
Breath time	≥1.6 s

Measurement accuracy

Standard deviation of repeatability with ethanol standard

Up to 0.50 mg/L	0.008 mg/L
≥0.50 mg/L	1.7 % of the measured value
Up to 1.00 ‰	0.017 ‰ or
≥1.00 ‰	1.7 % of the measured value depending on which measured value is higher

Sensitivity drift

Typically 0.4 % of the measured value per month

Calibration

every 12 months

Dimensions

50 x 133 x 29 mm

Weight

approx. 120 g

Power supply/battery

1 x 3 V CR123A	Battery in acc. with ; order no. 4543808
----------------	--

The batteries are designed for up to 600 measurements (depending on use and ambient conditions).

Protection class

IP 52	IP= International Protection (degree of protection) (5) Protected against dust in harmful quantities; complete contact protection (2) Protected against vertically dripping water with housing tilted up to 15 degrees
-------	--

Interface

Micro-USB port for Service only (do not use to charge the batteries)

CE marking

Electromagnetic compatibility

10 Annex

10.1 Glossary

Unit	Explanation
°C	degree Celsius; unit for temperature
°F	degree Fahrenheit; unit for temperature
hPa	Hectopascal; unit for ambient pressure
% r.h.	Percent relative humidity; relative atmospheric humidity
h	Unit for hours
mg/L	Milligram per litre, breath alcohol concentration
‰	Per mill; unit for blood alcohol concentration

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© **Dräger Safety AG & Co. KGaA**